

After each use: Wash and Dry on HOT. Do NOT microwave

DO NOT WASH FILTER.

To re-use filter, make damp and put in microwave TWO MINUTES.

How to Wear:

- 1) Insert filter, long side up.
- 2) Loop behind ears.
- 3) Shape nose piece around nose.
- 4) Pull tongue toward throat and secure snaps under chin.



This product has not been FDA cleared or approved. This product has been authorized by FDA under an EUA for use as source control by the general public as well as by HCP in healthcare settings as to help prevent the spread of infection or illness during the COVID-19 pandemic. This product is authorized only for the duration of the declaration that circumstances exist justifying the authorization of the emergency use of medical devices, including alternative products used as medical devices, during the COVID-19 outbreak, under section 564(b)(1) of the Act, 21 U.S.C. § 360bbb-3(b)(1) unless the authorization is terminated or revoked sooner.